MBL

CE

MESACUP Cardioliipin TEST

Cat. No. 7300E: 96 wells
Cat. No. 7300E: 96 puits
Art.-Nr. 7300E: 96 Auftragsstellen
Cat. No. 7300E: 96 pocillos
Cat. n. 7300E: 96 pozzetti
Cat. N° 7300E: 96 ποços
Αρ. Κατ. 7300E: 96 βυθισμάτων

MBL MEDICAL & BIOLOGICAL LABORATORIES CO., LTD.

5-10 Marunouchi 3 chome, Naka-ku, Nagoya, 460-0002 Japan

Tel: +81 52-971-2081 Fax : +81 52-971-2337 URL <http://www.mbl.co.jp>

- CONTENTS -
- CONTENU -
- INHALTSVERZEICHNIS -
- CONTENIDO -
- CONTENUTI -
- CONTENDO -
- ΠΕΡΙΕΧΟΜΕΝΑ -

English	1
Français	
Deutsch	9
Español	
Italiano	17
Português	
Ελληνικά	
Symbols/Symboles/Symbole/Símbolos/Simboli/ Simbolos/Σύμβολα/	26

English

INTENDED USE

The MESACUP Cardioliipin TEST is the quantitative enzyme-linked immunosorbent assay (ELISA) for the detection of anti Cardioliipin antibodies in human serum. The MESACUP Cardioliipin TEST is intended for in vitro diagnostic use as an aid in the determination of certain autoimmune diseases.

SUMMARY AND EXPLANATION

As well as biologically false positive serologic tests for syphilis (BFP) and lupus anticoagulant (LAC), anti-Cardioliipin antibodies are one of the anti-phospholipid antibodies. This antibody reacts with cardioliipin or complex composed of cardioliipin and cofactor (ex.β₂ glycoprotein 1) in human or animal sera. These antibodies are associated with thrombosis, organ infarct, recurrent fetal loss and thrombocytopenia. To describe patients who show the common features to these diseases, Hughes, Harris et al. introduced the new concept termed "anticardioliipin syndrome" in 1986. At present, it is widely recognized as "anti-phospholipid antibody syndrome". Anti-Cardioliipin antibodies can be measured by RIA or ELISA method. Since there were various methods and reference samples used to measure these antibodies in many laboratories, Harris's ELISA method were standardized in the International Work shop of 1986.

The MESACUP Cardioliipin TEST is for measuring with high sensitivity, anti-Cardioliipin antibodies present in the sera by ELISA.

PRINCIPLE

The MESACUP Cardioliipin TEST measures anti-Cardioliipin antibodies present in the serum by ELISA. Standards and patient sera are added to microwell coated with Cardioliipin, allowing anti-Cardioliipin antibodies to react with the immobilized antigen (Sample incubation). After wash to remove any unbound serum proteins, horseradish peroxidase conjugated anti human IgG monoclonal antibody is added and incubated (Conjugate incubation). Following another washing step, the peroxidase substrate is added and incubated for an additional period of time (Substrate incubation). Acid solution is then added to each well to terminate the enzyme reaction and to stabilize the color development. The assay can be quantified by measuring the reaction photometrically and plotting the results.

BRIEF ASSAY PROCEDURE

<Sample incubation>	Add 100 µl of diluted sample (1:101) to each well of microwell plate
(20-25°C) 60 min.	↓
	Wash
	↓
<Conjugate incubation>	Add 100 µl of conjugate solution to each well
(20-25°C) 60 min.	↓
	Wash
	↓
<Substrate incubation>	Add 100 µl of substrate to each well
(20-25°C) 30 min	↓
	Add 100 µl of stop solution to each well

↓
Read absorbance
↓
Interpretation of result

REAGENTS AND STORAGE

1) MICROWELL STRIPS

96 wells MICROWELLS STRIPS (8 x 12 wells) coated with Cardioliipin as antigen, the strips packaged in a strip holder and sealed in a foil envelope with desiccant, are stable at 2-8°C until the labeled expiration date.

2) Anti-Cardioliipin STANDARD SERUM (6 standards)

One vial containing 1.5 ml each of anti-Cardioliipin antibody positive human serum, PBS, bovine serum, 0.09% sodium azide. Anti-Cardioliipin concentration of each standard is on each label. Ready to use-make no further dilution. Stable at 2-8°C until labeled expiration date.

3) CONJUGATE REAGENT (101X)

One vial containing 0.3 ml of horseradish peroxidase conjugated mouse anti-human IgG (γ chain specific) monoclonal antibody, HEPES, Proclin 150 and BSA as 101X concentrate. Stable at 2-8 °C until labeled expiration date.

4) CONJUGATE DILUET*

One bottle containing 24ml of HEPES, Ptoclin 150 and BSA.

5) ASSAY DILUENT*

Two bottles containing 50mL of PBS, bovine serum, and 0.09% sodium azide. Ready to use. Stable at 2-8°C until labeled expiration date.

6) WASH CONCENTRATE (10X) *

One 100 ml bottle containing sodium dihydrogen phosphate, sodium chloride and Tween 20 as a 10X concentrate. Stable at 2-8°C until labeled expiration date.

7) SUBSTRATE SOLUTION*

One 20 ml bottle containing 3,3',5,5'-tetramethylbenzidine dihydrochloride/hydrogen peroxide (TMB/ H_2O_2). Ready to use. Stable at 2-8°C until labeled expiration date.

8) STOP SOLUTION*

One 20 ml bottle containing 1N Sulfuric acid. Ready to use. Stable at 2-8°C until labeled expiration date.

9) POSITIVE CONTROL SERUM

One vial containing 1.5 ml of anti-Cardioliipin antibody positive human serum with 0.09% sodium azide. Ready to use. Stable at 2-8°C until labeled expiration date.

10) NEGATIVE CONTROL SERUM

One vial containing 1.5 ml of anti-Cardioliipin antibody negative human serum with 0.09% sodium azide. Ready to use. Stable at 2-8°C until labeled expiration date.

*Those reagents can also be used for MESACUP Cardioliipin IgM.

PRECAUTIONS

- (1) This product is for in vitro diagnostic use only.
- (2) Do not use kit components beyond the stated expiration dates.
- (3) Avoid contact of reagents with eyes, skin and clothing. Reagents on skin must be washed away with plenty of water. TMB contains irritant and Stop Solution consists of a 1N sulfuric acid, which is a poison and corrosive.
- (4) Standards, Positive and Negative Controls are derived from human serum, in which HBs antigen, HCV antibody HIV-1 and HIV-2 antibodies has not been detected. No test method, however, can guarantee the absence of these or any other infectious agents. These reagents and all patient samples should be handled as if they are capable of transmitting AIDS, hepatitis or any other infectious diseases.
- (5) Standards, Positive Control, Negative Control and Assay Diluent contain sodium azide (0.09%) as a preservative and must be handled with caution - do not ingest or allow contact with skin or mucous membranes. Sodium azide may react with copper or lead in plumbing system to form explosive metal azides. Therefore, always flush with plenty of water when disposing materials containing sodium azide into a drain.
- (6) Some kit components contain animal origin materials, which are from non-infectious animals. These components, however, should be treated as potential biohazards in use and for disposal.
- (7) Matching lot numbers of Microwell strips, Standards must be used together in the assay. Do not substitute reagents from other kits.
- (8) All reagents must be brought to room temperature (20 -25°C) before starting the assay.
- (9) Do not expose the kit to direct sun during assay and storage.
- (10) Avoid microbial and cross contamination of reagents or samples.
- (11) Incubation temperatures above or below normal room temperature (20-25°C), shorter or longer time periods of incubation and inaccurate dilution may give erroneous results.
- (12) The wells must be rinsed with Wash Solution properly enough to avoid false positive.
- (13) Carefully pipette not to foam each sample and reagent to avoid cross contamination between microwells.
- (14) All microwell strips, which are not immediately required, should be returned to the zip lock pouch, which must be carefully resealed to avoid moisture absorption.
- (15) Wash concentrate may become turbid at 2-8°C, which does not cause inconsistent results.
- (16) Implement used for the test should be disposed or treated as shown below.

Soak in 2% final conc. glutaraldehyde solution for more than 1 hour or soak in 0.5% Sodium hypochlorite solution (available chlorine: approx. 5000ppm) for more than 1 hour or autoclave at 121°C for more than 20 minutes.
- (17) The Cardioliipin antibodies value obtained from this assay are an aid to diagnosis only. Each physician must interpret these results in light of the patient's history, physical findings, and other diagnostic procedure.

MATERIALS REQUIRED BUT NOT PROVIDED

- Microplate reader (wavelength: 450 nm, 620 nm/reference)

- Multichannel micropipette (e.g. 100 μ l - 300 μ l) for dispensing conjugate, substrate, and stop solution.
- Single channel pipette (10 μ l & 100 μ l)
- Reagent reservoir
- Autowasher or wash bottle
- Deionized or distilled water
- One liter graduated cylinder for preparation of wash solution
- Test tubes for patient sample dilutions (e.g. 1000 μ l)
- Disposable pipette tips
- Paper towels
- Basin and disinfectant
- Microplate cover

PROCEDURE

■ PREPARATION OF REAGENTS

- Bring all assay materials to room temperature (20-25 °C) prior to use.
- Microwell Strips: Remove required microwell strip from pouch and place them in the frame. Promptly return unused strips to refrigerated storage.
- Wash Solution: Prepare 1: 10 dilution of The Wash Concentrate prior to use.(ex. add 100 ml of Wash Concentrate to 900 ml of distilled water). The diluted wash solution is stable for 2 weeks at 2 -8°C.
- Conjugate Solution: The conjugate reagent must be diluted prior to use. Dilute Conjugate Reagent 1: 100 with Conjugate Diluent. (ex. add 0.1mL of Conjugate Reagent to 9.9mL of Conjugate Diluent.)
- Do not dilute Standards, Negative Control, Positive Control, Assay Diluent, Substrate and Stop Solution, which are ready-to-use.

■ PREPARATION OF SAMPLES

- Dilute each patient serum 1:101 by adding 10 μ l of serum to 1mL of Assay Diluent.
 - *Diluted samples must be used within a day.
 - *Diluted samples can be also used for MESACUP Cardioliipin IgM.
- Use fresh patient sera. If storage is needed, they should be frozen below -20°C for up to one month, below -70°C for longer storage. Do not repeat freezing and thawing.
 - *In case stored below -20°C for more than 6 months or freezing and thawing repeatedly, nonspecific results are obtained because of IgG denaturation.

■ ASSAY PROCEDURE

STEP 1. (SAMPLE INCUBATION)

Using the multi-channel pipetor, transfer 100 μ l of Standards, each diluted sample, Positive and Negative Controls into the appropriate microwells of the antigen test plate.

- * Incubation starts on pipetting to the antigen-coated microwells. Pipetting should be completed as quickly as possible.

Cover wells with a plate sealer and incubate for 60 minutes at room temperature (20-25 °C).

STEP 2. (WASHING)

Aspirate or discard the well contents. Fill the well with Wash Solution and then completely aspirate or discard the contents. Wash 4 times. Tap the plate on a paper towel to remove any remaining Wash Solution. When autowasher is used, wash 4 times.

* Each laboratory is recommended to confirm its own appropriate washing times and set-up.

* Wash Solution should be used at 20-25 °C.

STEP 3. (CONJUGATE INCUBATION)

Pour Conjugated Reagent into a reservoir. Add 100 µl of the Conjugated Reagent to each well with multichannel pipette. Cover wells with the plate sealer and incubate for 60 minutes at room temperature (20-25°C).

STEP 4. (WASHING)

Wash the microplate following the STEP 2 procedure.

STEP 5. (SUBSTRATE INCUBATION)

Pour Substrate into a reservoir and pipette 100 µl of the Substrate to each well with multichannel pipette.

* The reservoir should be different from the one, which was used for pouring conjugate solution. A new disposable reservoir should be used because Substrate is easily oxidized by metal ion.

* The Substrate, once poured in a reservoir, should not be returned to the bottle.

Cover wells with the plate sealer and incubate for 30 minutes at room temperature (20-25°C).

STEP 6. (STOP REACTION)

Pour Stop Solution into a reservoir. Pipette 100 µl of the solution to each well with multichannel pipette.

■ READING

Read the absorbance of each well at 450 nm. If a dual wave length plate reader is available, set the test wavelength at 450 nm and the reference at 620 nm.

* Reading should be done as quickly as possible after stopping the reaction.

* Ensure that the bottom of the plate is clean and dry, and that no air bubbles are present on the surface of the liquid in the wells before reading the plate.

■ CALCULATION OF RESULT

Calculate the mean absorbance value of each standard and plot against log standard concentration on suitable graph paper. The concentration of the samples can then be read from the standard curve. Alternatively a suitable computer and curve-fitting program may be used.

*The assay is standardized by reference to the standard product (unit: GPL) for ELISA method established by E. Nigel Harris et al.

■ QUALITY CONTROL

Each assay result should meet the following criteria

A₄₅₀ of anti-Cardiolipin Standard-1: ≤ 0.09

A₄₅₀ of anti-Cardiolipin Standard-6: 1.0 – 2.0

The Positive and Negative Controls must give the following results:

	Positive Control	Negative Control
Anti-Cardiolipin value (U/ml)	≥20	<10

If any of these are not met, the results are invalid and the test should be repeated.

Before repeating assay, check the following procedure.

- Incubation Temperature
- Incubation Period of Time

- Washing

* When the quantitative value exceeds 120U/ml, it should be interpreted as greater than 120U/ml or the test must be repeated after diluting the sample.

TEST INTERPRETATION AND EXPECTED VALUES

The following is intended only as a guide for interpretation. Each laboratory is recommended to establish its own criteria for test interpretation based on sample populations typically encountered.

Anti-Cardiolipin value	Interpretation
< 10 U/ml	Negative for anti-Cardiolipin Ab
Greater than or equal to 10 U/ml	Positive for antiCardiolipin Ab

* When the MESACUP Cardiolipin TEST assayed serum samples from 205 healthy donors, the maximum value of 95.0% tile is less than 10 U/ml.

■ LIMITATIONS

As with other diagnostic test procedures, the results obtained with the MESACUP Cardiolipin TEST serve only as an aid to diagnosis and should not be interpreted as diagnostics in themselves.

PERFORMANCE CHARACTERISTICS

■ CLINICAL SPECIFICITY AND SENSITIVITY

Disease	Positive sample	Positive rate
APS	84/112	75.0%
SLE	94/213	44.1%
ITP	9/19	47.4%
Other diseases	47/252	18.7%
Normal Serum	10/205	4.9%

APS: anti-phospholipid antibody syndrome

SLE: systemic lupus erythematosus

ITP: idiopathic thrombocytopenic purpura

■ PRECISION

Repeatability was demonstrated by testing 3 samples in sextuple. Reproducibility was determined by testing 3 samples on 5 different days (day-to-day) and by testing 3 samples in 3 lots (lot-to-lot). %CV values for reproducibility and repeatability were below 15% for each sample.

■ ASSAY RANGE

The assay range of this kit is up to 120 U/ml.

■ INTERFERING SUBSTANCES

Intrafat (more than 1.5%) , Ascorbic Acid (more than 30mg/ml), Hemoglobin (more than 600 mg/dl) and/or Bilirubin (more than 8.0mg/dl) may be affective on the assay result. Avoid using highly hemolysed samples or highly lipemic samples.

REFERENCES

1. Harris E.N., et al: Evaluation of the anti-cardiolipin antibody test: report of an international workshop held 4 April 1986. *Clin EXP. Immunol* 68: 215-222 (1987)
2. Hughes G.R.V., et al: The Anticardiolipin Syndrome. *J. Rheumatol.* 13: 486-489 (1986)

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